a977

ambition

- sporty, young design
- Hybrid Memory Hinge™ for automatic and exceptionally comfortable fit
- Performance Steel™ temple inlays
- Traction Grip™ for supreme wearing comfort
- Temples can be shortened easily and adjusted to different head shapes
- Also wearable as sunglasses (with tinted lenses)
- Frame comes in its own hardcase incl. adidas cleaning cloth

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spare parts a977

BC 606 + GK219	temple (125 mm) + AC logo part
GK 219	temple tips
SR 360/RH	temple joint screw
SP 106	soft nose pads
SP 106-1	extended soft nose pads
SP 108	hard, transp. nose pad
STG 70	showing lens 5 base
Ma977	front with side parts
PL 13	adidas cleaning cloth
SHE 166	Hardcase
FK 78-3	box

BC 606 + AC 317, GK 219

SR 360/RH

Product information

General glazing information:

- The distance between the peak of the bevel and the rear surface of the lens in the nasal area should not exceed 5.00 mm so as not to impair the function of the adjustable nose pad (see fig. 1 + 2)

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Gabarit de lentille disponible.

Options for adjusting the eyewear to individual wearers:

Inclination

- Inclinable metal hinge (note: only use parallel action pliers with plastic jaws; see fig. 3)

Temples

- marks on the inside of the temples (5 mm spacing) make it easier to shorten the temples (see figures 4 - 8)
 - the temples can be adjusted to the shape of the head and ears (see figures 9 - 10)
 - temples are available as spare parts

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